


RKR Plunger Pedal Lock



Fitting Instructions


1. Insert M10 x 40 Bolt through unused hole on side of base
2. Place spacer, yellow bar and then spring washer on to bolt.
3. Tighten with nyloc nut

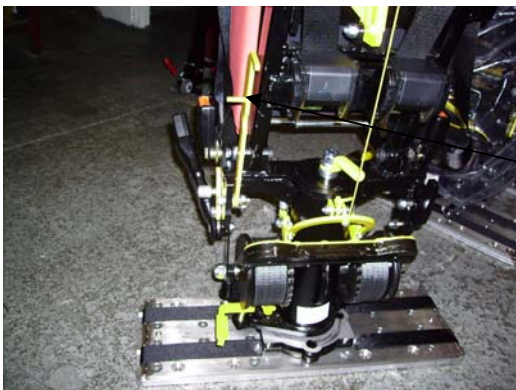

How to Use

Unlocked

Unlocked position: Swivel the yellow bar upwards so hook is at the top.

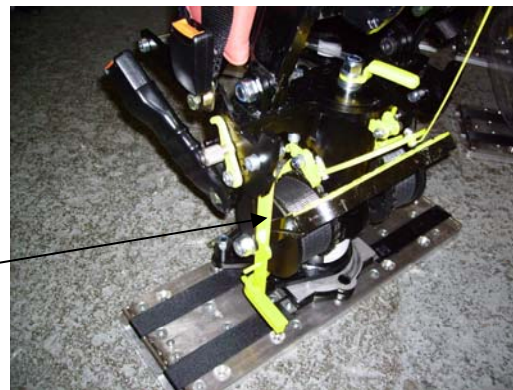

Locked

Locked position: Swivel the yellow bar with hook facing downwards under the yellow pedal. This prevents any movement of pedal.

Note: Plunger pedal lock should always be in locked position when pedal is not being used to rotate seat.