

## Instruction Sheets

### 4Point Tie Down

Z044/Z045: Z044/K / Z045/K

1. Position wheelchair either between or over two floor tracks (of 330mm centres) and apply brakes.

2. **Front Wheels:** Wrap the fixed length restraints around the wheelchair frame above the front wheels and snap the tongues into the buckles or snap on the karabiners.

Locate the studs into track in front of the wheelchair.

3. **Back Wheels:** Wrap the adjustable length restraints around the wheelchair frame near the rear wheels and snap tongues into buckles or snap on the karabiners.

Make sure the gold coloured over centre buckles are open. Locate the studs into the tracks at the rear of the wheelchair.

Pull the D-ring strap tight until slackness is eliminated. Close over centre buckle for final tensioning. Place the D-ring strap onto the attached Velcro so that its is not loose on the vehicle floor.

The wheelchair is now fixed between four points.

4. **Unlock:** Press down the push button buckles or unhook the karabiners on each of the 4 restraints to release the wheelchair.

**NOTE:** This equipment only secures the wheelchair. The wheelchair passenger must be secured independently.

