

## Instruction Sheets

### **B056/B057: NMI Double Inertia Safety System**

#### **Step 1: To Mount Reel & Stalk Fixtures to Floor Track**

Ensure the spring loaded plunger and clamping screw on base plate are in the unlocked (raised) positions. Place into floor track just behind wheelchair seat position.

Turn large knob to release locator and slide fixture along the track until the plunger engages into the track. Screw down clamp to lock fixture into the track.

#### **Step 2: To Secure Passenger**

Pull front inertia reel up and across lap of passenger, passing strap through sides of wheelchair for maximum security. Fit tongue of strap into stalk buckle.

Similarly, pull rear strap of inertia reel up and over shoulders, diagonally down across the chest and fit to the other stalk buckle.

The comfort clips may now be positioned and locked against the reel bodies so as to relieve tension on the passenger.

#### **Step 3: To Release Passenger**

Open comfort clips on both straps. Release straps from stalk buckles. Hold straps until fully retracted into reels to avoid whiplash.

**NOTE: This equipment only secures the wheelchair passenger. The wheelchair must be secured independently.**

