



Instruction Sheets

M056/3 / M057/3: Millennium Double Inertia Safety System

Step 1: To Mount Reel & Stalk Fitting to Floor Track

Move the red safety lock nut along the slot from position A to position B. Whilst still holding the nut in position B pull back the red handle. The Millennium fitting is now in the unlocked position.

To lock the Millennium Fitting into the tracking simply push forward the red locking handle to its furthest point. The red safety lock will automatically drop into position A in the slot as shown below.

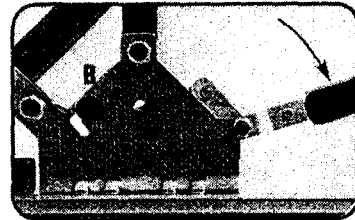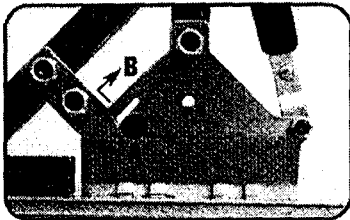

Step 2: To Secure Passenger

Pull front inertia reel up and across lap of passenger, passing strap through sides of wheelchair for maximum security. Fit tongue of strap into stalk buckle.

Similarly, pull rear strap of inertia reel up and over shoulders, diagonally down across the chest and fit to the other stalk buckle.

The comfort clips may now be positioned and locked against the reel bodies so as to relieve tension on the passenger.

Step 3: To Release Passenger

Open comfort clips on both straps. Release straps from stalk buckles. Hold straps until fully retracted into reels to avoid whiplash.

NOTE: This equipment only secures the wheelchair passenger. The wheelchair must be secured independently.

