

Instruction Sheets MITS

INSTRUCTION SHEET

MILLENNIUM INCUBATOR TRANSPORTATION SYSTEM (MITS)

PLACE THE MILLENNIUM FITTING IN THE TRACKING AND LOCK IN PLACE.

RELEASE THE OVER-CENTRE BUCKLE AND EXTEND THE ORANGE WEBBING.

RELEASE THE ORANGE PUSH BUTTON BUCKLE AND LOOP THE WEBBING AROUND THE MAIN BODY OF THE TROLLEY.

CLICK THE TONGUE BACK INTO THE ORANGE PUSH BUTTON BUCKLE AND ENSURE IT IS LOCKED.

USING THE D RING TAKE UP THE SLACK IN THE WEBBING UNTIL RESISTANCE IS FELT.

LOCK THE OVER-CENTRE BUCKLE.

CHECK ASSEMBLY FOR SECURITY AND REPEAT ON EACH CORNER.

