



RKR COMPETENCY FORM **ROTATING KING RIPS**

When cascading familiarisation training of the RKR to others, please ensure that the following tick checklist is observed.

It is important that person being familiarised actually operates the product himself or herself.

Start with the RKR in the seat position.

REMOVE SEAT CUSHION

UNLOCK THE SEAT TUBE FROM THE BASE FLOOR PLATE

ROTATE 90 DEGREES AND SLIDE TO WALL

Now place the unit in its position to lock the wheelchair.

PULL BUNGEE OFF EXTENDING BACK

OPEN THE EXTENDING BACK

FIT ADAPTER BEAM INTO UNIT

RELEASE THE YELLOW DOGS OFF THE DRUMS

PULL OUT THE WEBBING AND ATTACH TO WHEELCHAIR

RE ENGAGE THE YELLOW DOGS ONTO THE DRUMS

RATCHET WHEELCHAIR TIGHT

LOWER SAFETY LOCK ON HORSESHOE
AND SWING DOWN SIDE PEDAL ARM

FIT THE OCCUPANT BELTS

STORE CUSHION ON THE BACK

Refer to NMI RKR Instruction sheets for detailed training notes.

Sept 06