


# NMI Millennium Combined Wheelchair & Occupant Belts: User Instruction Leaflet


To be operated by trained personnel only

## Setting up and applying the webbing to secure the wheelchair


### 1: Attaching Fixed Straps to the front of the wheelchair

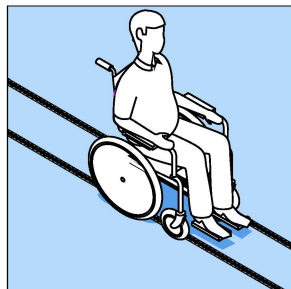


Position the wheelchair where the tracking is set at the measured centres of 330mm

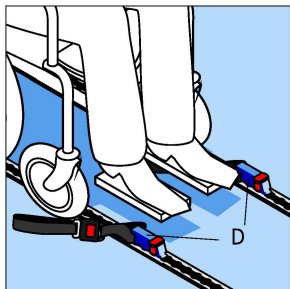

Lock Plunger Stud Fittings D (see Note 1 on next page to see how to do this) of Fixed Straps into tracking in front of the wheelchair

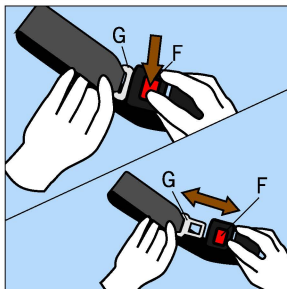


Separate belt tongue G and buckle by pressing button F while pulling apart

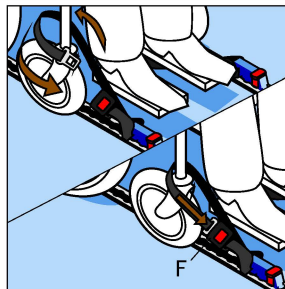


Wrap around appropriate attachment point on front of wheelchair and reconnect with buckle F. Repeat process on other side of wheelchair


### 2: Attaching Adjustable Straps to the rear of the wheelchair and securing it from movement

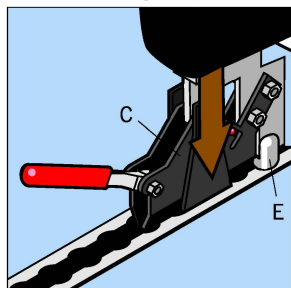


Align locator E with a hole in the floor tracking then insert the whole Millennium 2 Fitting C down into the tracking

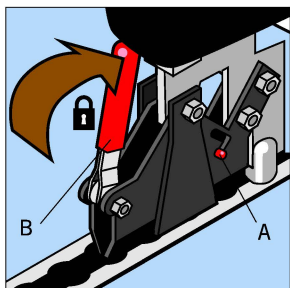


Close handle B to lock the chair in its new position. Safety bolt A should rest in their lower position. The device is now secure

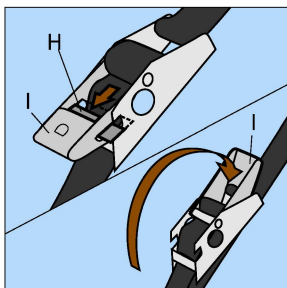


To make the belt adjustable push in H of I and rotate I to the other end

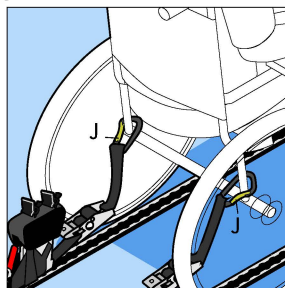


Attach karabiners (J) to manufacturer's recommended attachment points at the back of the wheelchair

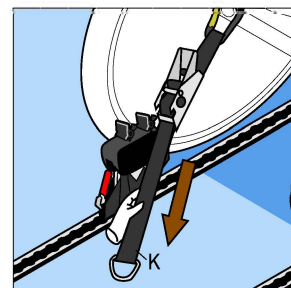

Pull loose ended strap K till webbing is tight and so strap in front of the wheelchair is also in tension

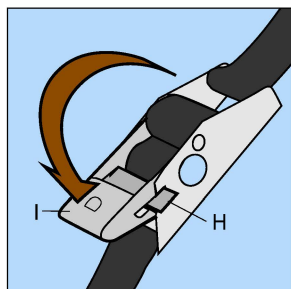

Close I till H engages so the strap becomes fixed. Repeat tensioning process on other side of wheelchair to secure it. **The wheelchair should not be able to move**

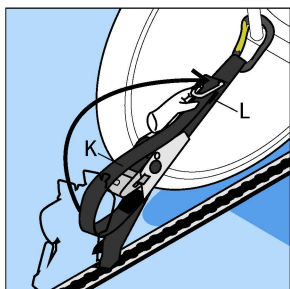

Attach the loose end of the strap K to the Velcro patch L. Repeat on other strap



**REMEMBER**  
**You have only secured the wheelchair.**  
**The wheelchair occupant must be secured independently.**  
(See next page)

Continued on next page...

## ...continued: Applying the passenger safety belts

### 3: Applying the passenger safety belts

Pull strap nearest the wheelchair up and across lap of passenger, passing strap through sides of wheelchair for maximum security and put into a stalk buckle


Pull the remaining strap up and over shoulder, diagonally down across chest and into the other stalk buckle. (See right image) If the combined restraint is supplied with an additional tongue on the shoulder belt then this should be used in conjunction with an extension belt. Where applicable, the extension belt clips into the inertia tongue and goes from the shoulder and locks into the vehicle wall. (See image on far right)

Adjust the comfort clips so they are positioned and locked against the casing. This alleviates downward pressure on the occupant's shoulder and stops the belt cutting across the neck (see Note 2 at the bottom of the page)


The wheelchair and occupant should now be secure.

**For the safest method of travel** - This safety system should be used in conjunction with the NMI Saferider back & headrest protection


Without extra shoulder strap tongue

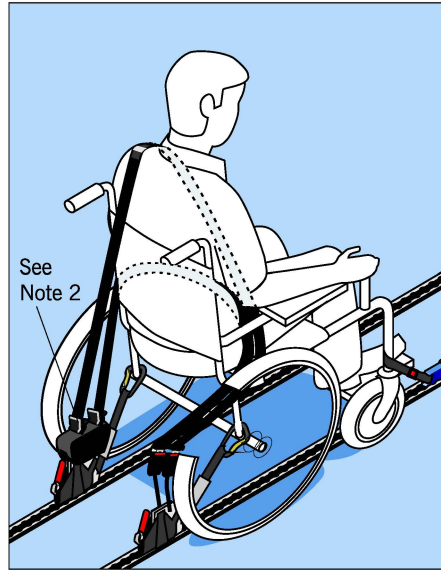


With extra shoulder strap tongue

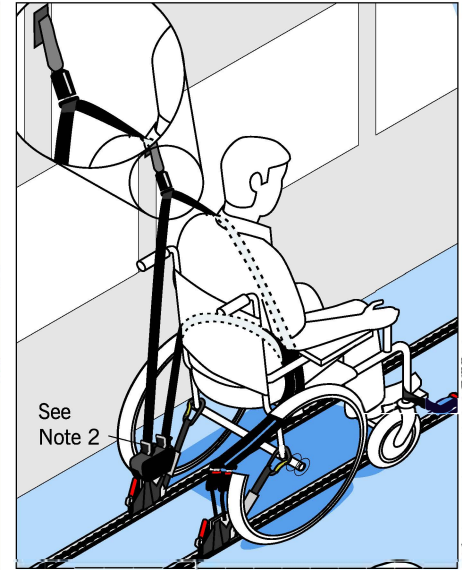


## Releasing the passenger & removing the restraint devices

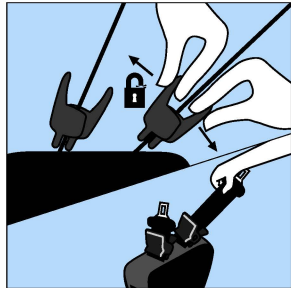


To release the passenger open comfort clip on both straps. Release straps from stalk buckles and **hold straps until they retract into the reels to avoid whiplash**

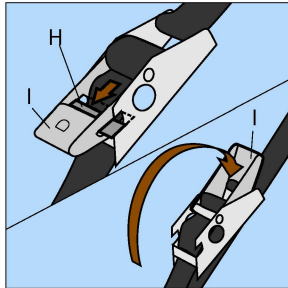

To make the rear wheelchair straps adjustable push in H of I and rotate I to the other end. The karabiners can now be removed from the wheelchair. At the front of the wheelchair press buckle buttons on fixed straps to release tongues

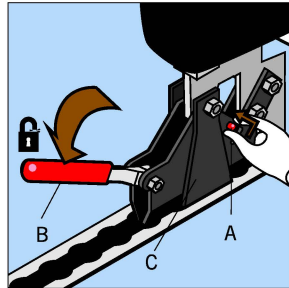


While holding safety bolt A in its upper position pull handle B away from the Millennium 2 fitting C to unlock. The restraint device is now unlocked and can be removed from the floor tracking by lifting it out

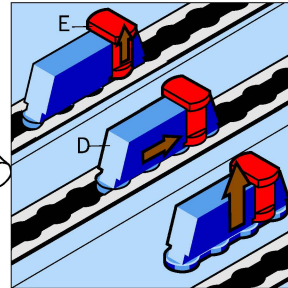


To remove front fixed strap Plunger Stud Fittings pull up Plunger E, slide Plunger Stud Fitting D till the unit aligns with the holes in the track. When aligned the device can be lifted out

## Notes

### Note 1: Plunger Stud Fitting installation

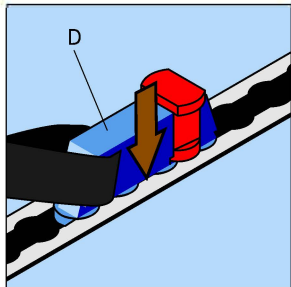


Attach the webbing devices to the floor by inserting the Plunger Stud Fitting (D) into the floor tracking

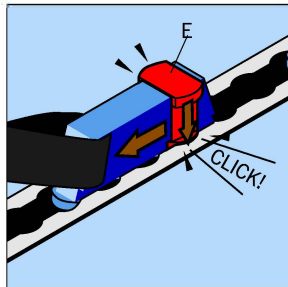


Slide the Plunger Stud Fitting until plunger E drops into a hole in the floor tracking. The device is now locked in the floor tracking

### Note 2: Adjust comfort clips

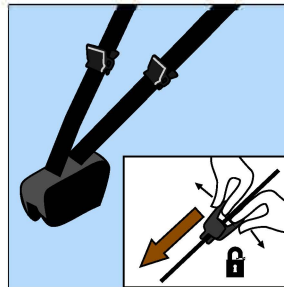

Unlock comfort clips by pulling the two larger ends apart.

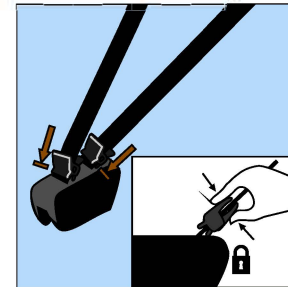

Once clips are at the bottom they can be locked by pushing the two larger ends together. These will make the belt straps feel less tight for the user