


NMI Millennium Seat Back and Ratchet: User Instruction Leaflet


To be operated by trained personnel only

Securing the wheelchair and seat back


1: Securing the seat back in the tracking

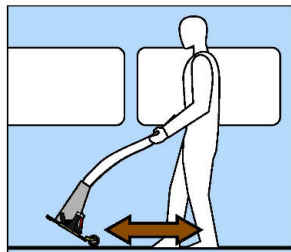


Tilt the seat back and use the wheels to roll seat into correct position over the tracking

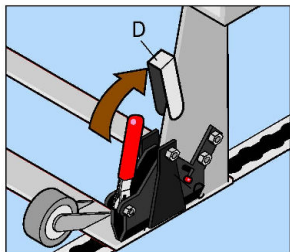


To unlock the Millennium 2 Fitting C first remove guard D. Repeat on other leg

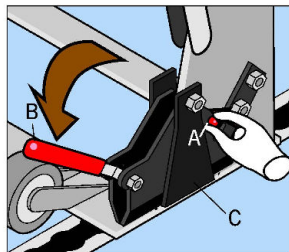


While holding safety bolt A in its upper position pull clamp B away from the Millennium 2 fitting C to unlock. Repeat on other leg

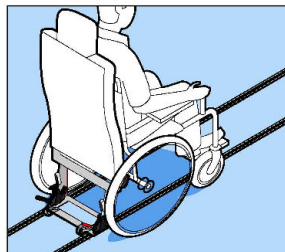


Insert the seat back into the tracking behind the wheelchair. Ensure front peg is in the hole and that the base is sitting flat on the tracking

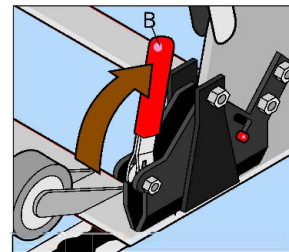

Close handle B to lock the seat back in position. Repeat on other side of seat

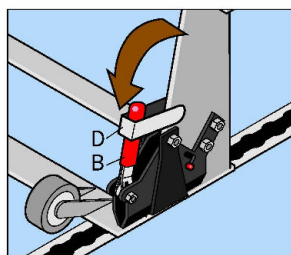

Replace the guard D over B. Repeat on other leg. The seat back is now secure

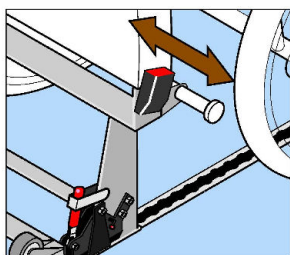


The adjustable bars can be slid to the appropriate position to meet the wheelchair frame. Repeat on the other side of the seat




Caution: Before Securing the wheelchair ensure that the seat is rigid in the base

2: Securing the wheelchair

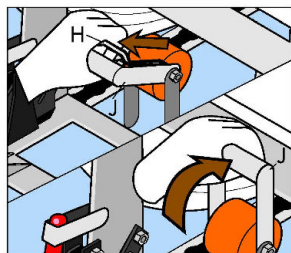


To open the ratchet squeeze bar H to unlock and lift handle J up and forward to uppermost point and then let go

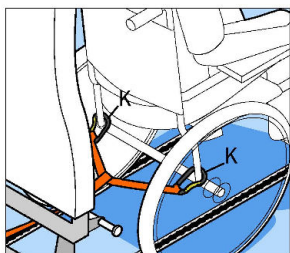


Pull the karabiners (K) from the front of the seat base and attach to the rear of the wheelchair frame as low down as possible on the wheelchair manufacturer's recommended securement points




Caution: Under no circumstances must the karabiners be attached to the front of the wheelchair frame

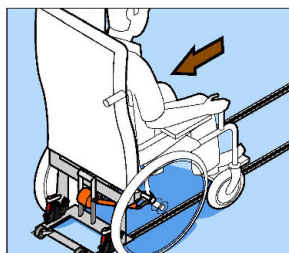

Reverse the wheelchair against the seat back. Ensure the adjustable bars are against the frame

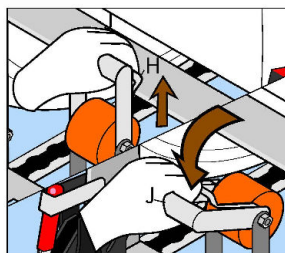

To engage ratchet squeeze bar H and lower handle J down midway and then let go

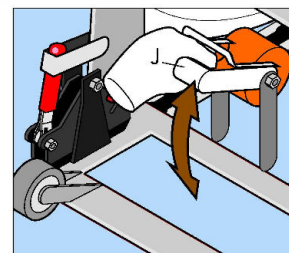

Ratchet the wheelchair in tight by holding main handle J and moving it up and down. When webbing is secure lower handle J to its lowest position



Check there is no slack in the webbing



Check wheelchair frame is against adjustable bars



This product should only be used with manual wheelchairs


3: Securing the passenger



Secure the passenger using the two all age inertia belts. Take one all age inertia belt and insert it into the opposite buckle



Repeat the previous step using the other all age inertia belt and fixing it into the opposite buckle on the other side of the seat

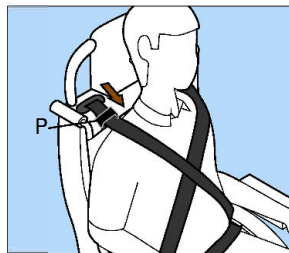

Position the age adjuster P (on both shoulder straps) to ensure webbing does not cut across the occupants neck