


# NMI 4 Point Webbing: User Instruction Leaflet


To be operated by trained personnel only

## Setting up and applying the NMI 4 Point Webbing to secure the wheelchair


### 1: Attaching Fixed Straps to the front of the wheelchair

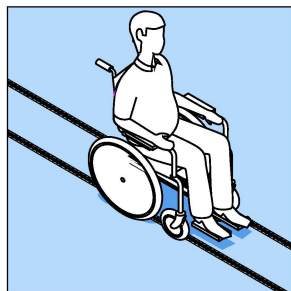


Position the wheelchair with the floor tracking

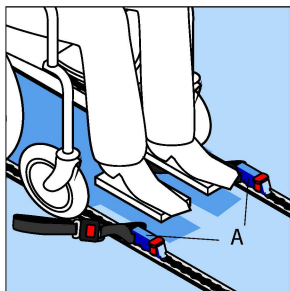

Lock Plunger Stud Fittings A (see Note 1 below to see how to do this) of Fixed Straps into tracking in front of the wheelchair

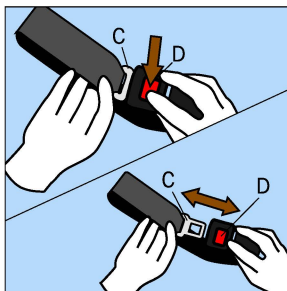


Separate belt tongue C and buckle by pressing button D while pulling apart

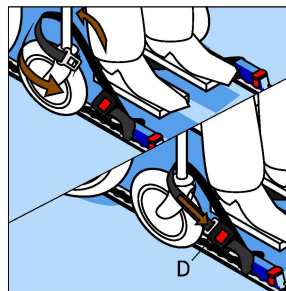


Wrap around appropriate attachment point on front of wheelchair and reconnect with buckle D. Repeat process on other side of wheelchair


### 2: Attaching Adjustable Straps to the rear of the wheelchair and securing it from movement

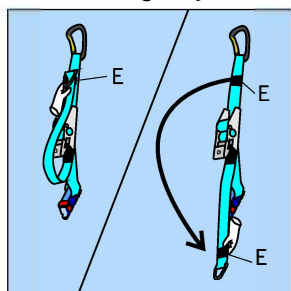


Separate ends of the adjustable strap by pulling velcro patches E apart

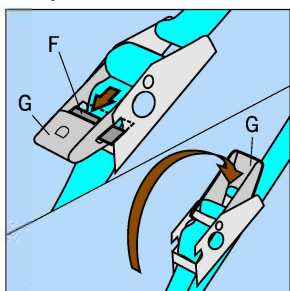


To make the belt adjustable push in F of G and rotate G to the other end

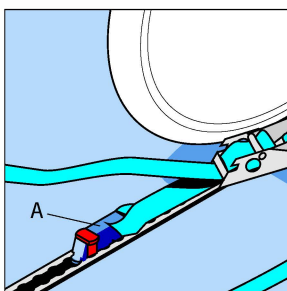


Insert Plunger Stud Fitting A into tracking behind the wheelchair

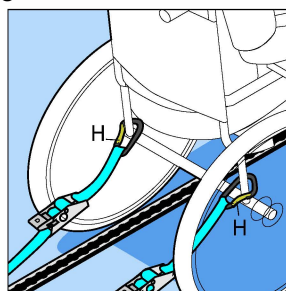


Attach karabiners (H) to manufacturer's recommended attachment points at the back of the wheelchair

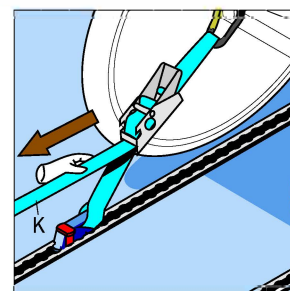

Pull loose ended strap K till webbing is tight and so strap in front of the wheelchair is also in tension

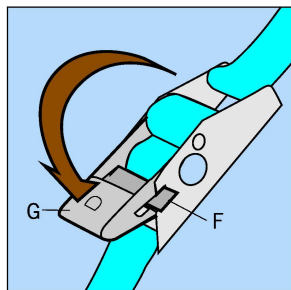

Close G till F engages so the strap becomes fixed. Repeat tensioning process on other side of wheelchair to secure it. **The wheelchair should not be able to move**

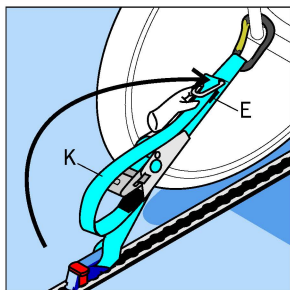


Reattach the loose end of the strap to the Velcro patch. Repeat on other strap




#### REMEMBER

**You have only secured the wheelchair.  
The wheelchair occupant must be secured independently.**


### Note 1: Plunger Stud Fitting installation/removal

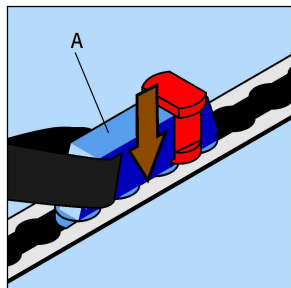

Attach the webbing devices to the floor by Inserting the Plunger Stud Fitting (A) into the floor tracking

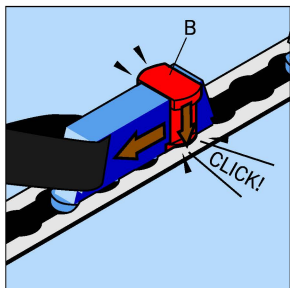

Slide the Plunger Stud Fitting until plunger B drops into a hole in the floor tracking. The device is now locked in the floor tracking

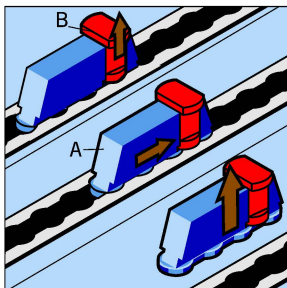

To remove pull up Plunger B, slide Plunger Stud Fitting A till the unit aligns with the holes in the track. When aligned the device can be lifted out